


freeflex® Instructions for Use



Check the **freeflex** solution composition, lot number, and expiry date. Using the pre-cut corner tabs, peel open the overwrap and remove the primary container. Inspect the container for leakage by squeezing firmly. If leaks are found, visible particles or cloudy, do not use.

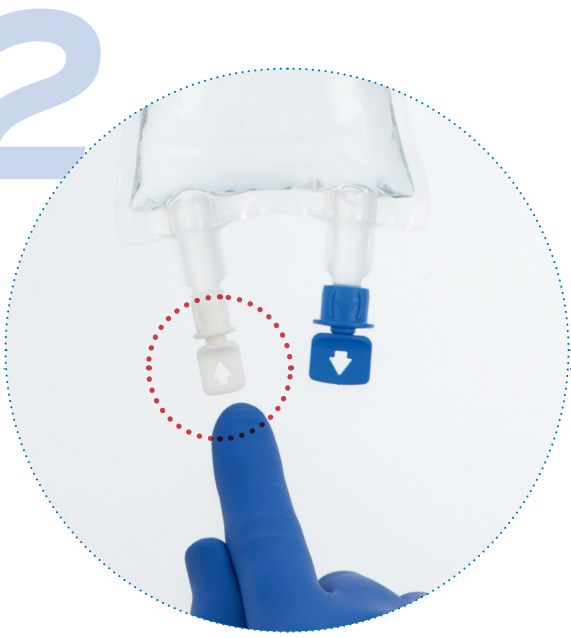

Identify the WHITE additive port with arrow pointing toward container. If no additive medication is required, go to step 5.



Immediately before injecting additive medication, break off the WHITE additive port cap.



Hold the base of the WHITE additive port horizontally. Insert needle horizontally through the center of the WHITE additive port's septum and inject additives. Mix container thoroughly.



Identify the BLUE infusion port with arrow pointing away from container.



Immediately before inserting the infusion set, break off the BLUE infusion port cap. Use a non-vented infusion set or on a vented set, close the air inlet. Use a spike with diameter as specified in ISO 8536-4, 5.6 +/- 0.1mm.



Follow the instructions for use for the infusion set. While holding the base of the infusion port, insert the spike parallel with the infusion port and rotate your wrist slightly until the spike is inserted.



Hang the bag on the infusion stand.

Consult a pharmacist for additive medication compatibility.

freeflex®
All you need in a bag

Please refer to the package insert for the applicable product for a complete list of all precautions and warnings related to **freeflex**

Fresenius Kabi USA, LLC
Three Corporate Drive,
Lake Zurich, IL 60047
Phone: 1.888.386.1300
www.fresenius-kabi.us

**FRESENIUS
KABI**
caring for life